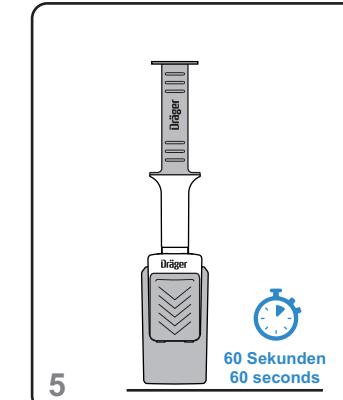
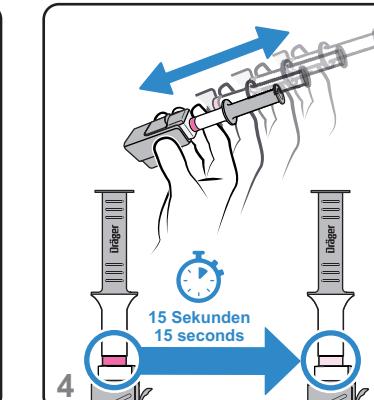
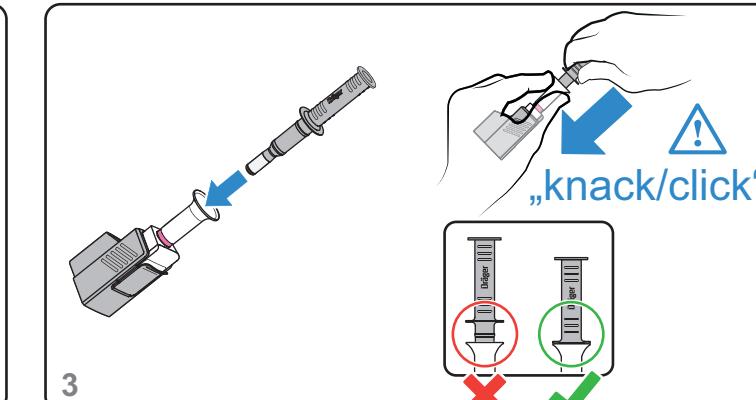
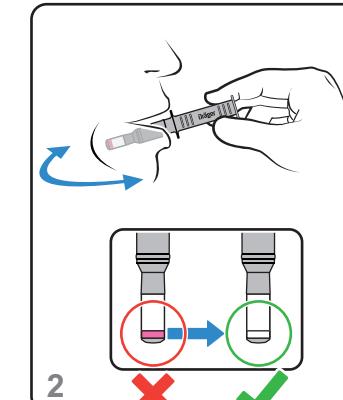
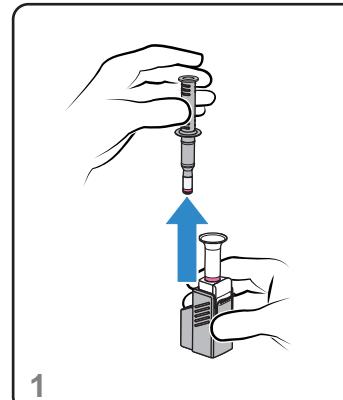
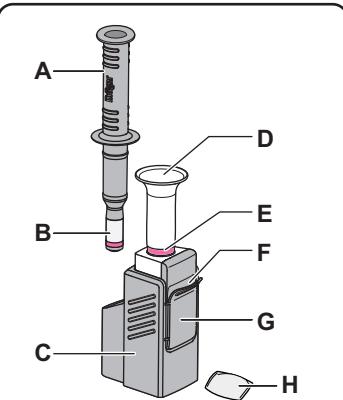
Scan mich
Scan me

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Requirements for a successful test:

- Read the following instructions in full.
- Keep a watch with a second indicator to hand.
- Do not consume any food or drink for 10 minutes before sampling.
- The test should be carried out at an ambient temperature of between +5 °C and +40 °C (+41 °F and +104 °F).
- Do not open the foil pouch until immediately before using the test.

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Contents:
A - sample collector
B - mouthpiece with color indicator
C - test cassette
D - funnel
E - indicator ring
F - test strips
G - security seal
H - desiccant bag
 (Not required for the test, dispose of in a proper manner)

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1. Grasp the sample collector (**A**) by the handle and pull it firmly out of the test cassette (**C**) against resistance.
Do not touch the mouthpiece (B**)!**

2. Place the sample collector in your mouth.
Move the sample collector back and forth in your cheek pouch.
Switch from one side of your mouth to the other.
This process takes approx. 15 seconds.
You can stop sampling as soon as the red color indicator on the mouthpiece of the sample collector loses its color.
Do not suck or chew on the sample collector!

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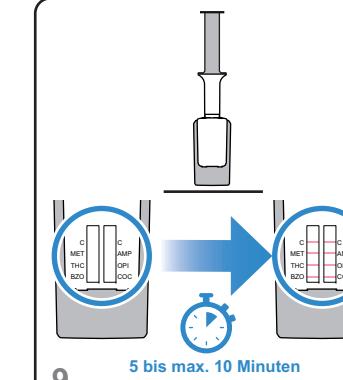
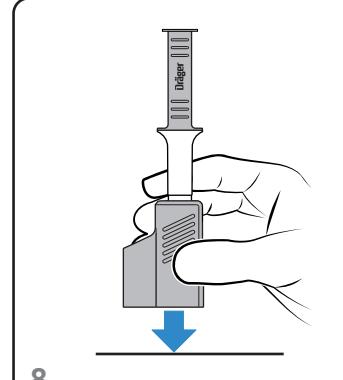
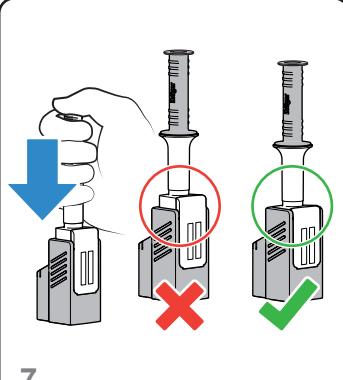
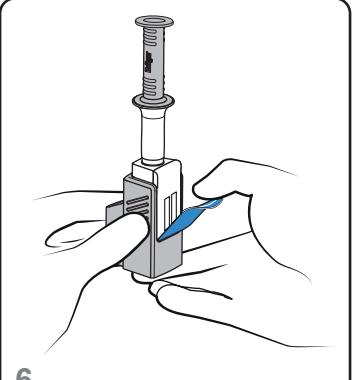
5. Place the test kit on an even surface for **60 seconds**.

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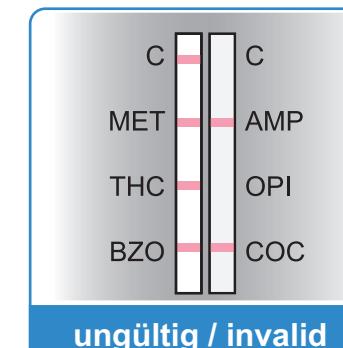
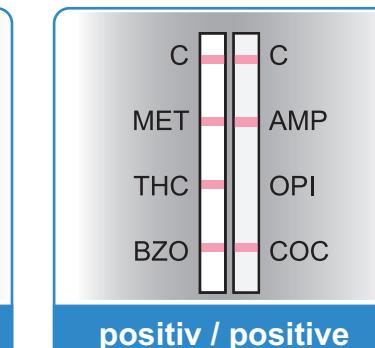
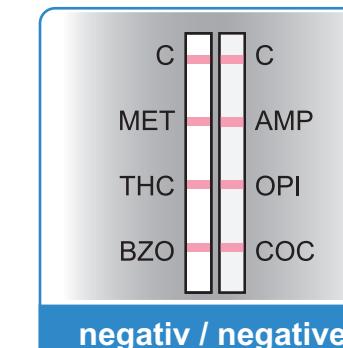
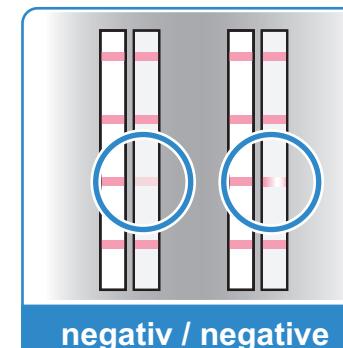
6. Fully break the security seal (**G**).

7. Grasp the sample collector (**A**) and press the funnel (**D**) down firmly in the test cassette (**C**).
You will hear a click.
The test liquid runs upward on the test strip (**F**).

8. If the test liquid does not run upward on the test strip within one minute, lightly tap the test kit with the stand on an even surface.

9. Place the test kit on an even surface.
Wait 5 minutes before reading the result. Do not read after 10 minutes.
Results that are read after 10 minutes may be incorrect.

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i Even faint, partially colored, or broken lines should be evaluated as “negative”.

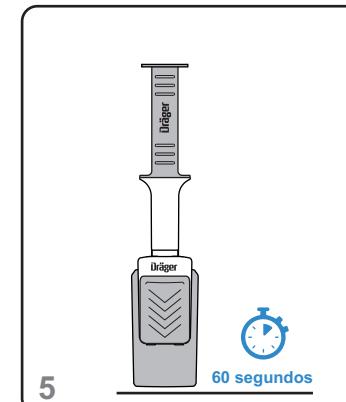
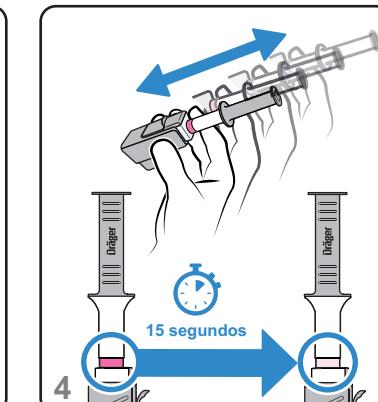
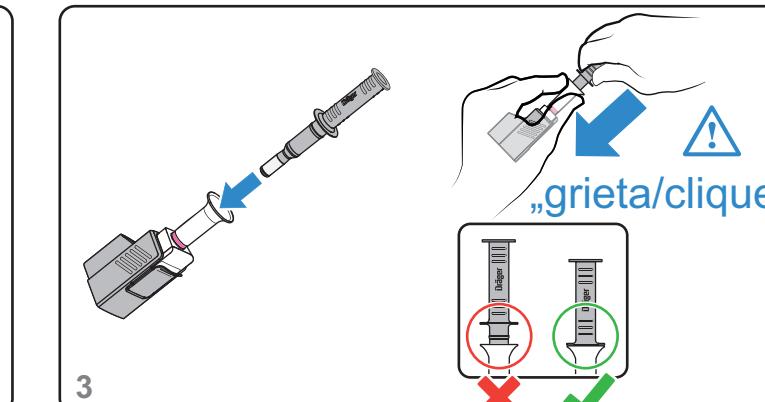
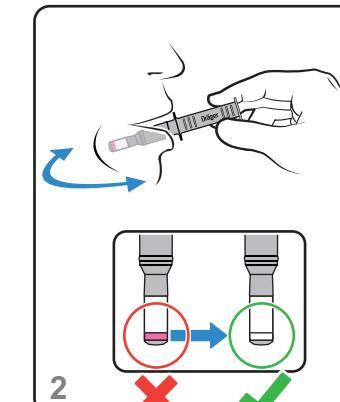
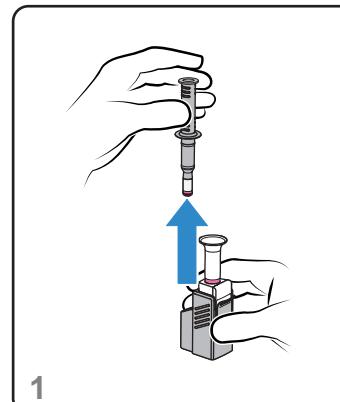
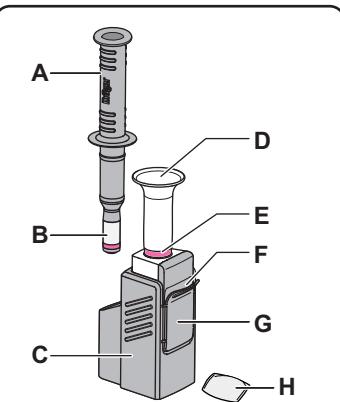
Valid negative result:
Both control lines (**C**) appear, no consumption of
 - methamphetamines (MET)
 - amphetamines (AMP)
 - cannabis (THC)
 - opiates (OPI)
 - benzodiazepines (BZO)
 - cocaine (COC).

Valid positive result:
Both control lines (**C**) appear, indicating the consumption of opiates.

Invalid result:
If no control lines (**C**) appear, the test is invalid.
If one control line (**C**) does not appear, the test is invalid for this test strip.
The test is only valid for the left test strip, and the result is negative.
The right test strip is invalid.

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